




# Usage data collection made easier with MyLink™


## About MyLink

MyLink enables you to easily upload your Optune Gio™ device data to the remote storage server from the comfort of your home.

## MyLink and its indicators next to the Optune Gio device



-  On/off indicator
-  Downloading treatment data indicator
-  Uploading treatment data indicator

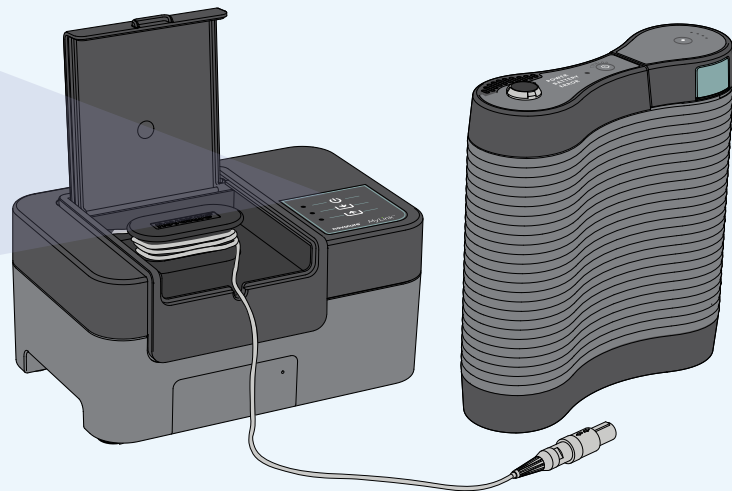

MyLink

Optune Gio device

**WARNING!**—All servicing procedures must be performed by qualified and trained personnel. Attempting to open and service the system may result in an electric shock by touching the inner parts of the equipment. It could also cause damage to the system.

**WARNING!**—No modification of this equipment is allowed.

Note: The unit is using a cellular network for its operation. If you are trying to use it in a non-cellular reception environment, the unit will not work properly.

MyLink is to be used with Optune Gio model TFH9100 only.

## Get 24/7 personalized support




Call MyNovocure™ at  
**1-855-281-9301**



Email MyNovocure at  
**[support@novocure.com](mailto:support@novocure.com)**


Translation is available in over 240 languages.


# Using your MyLink™

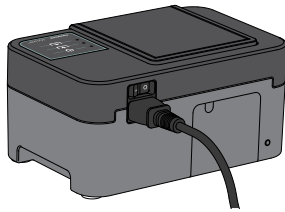

MyLink

1

Plug the **MyLink modem** into the wall, then **turn MyLink on** with the switch in the back. The top  light will blink, then turn solid green.

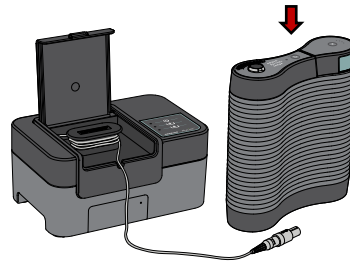

MyLink


Optune Gio™  
device


2

Connect the **gray MyLink cable** to the top of the Optune Gio device.



3


Turn on the **Optune Gio device** to start the download step. The middle  light will blink while downloading.

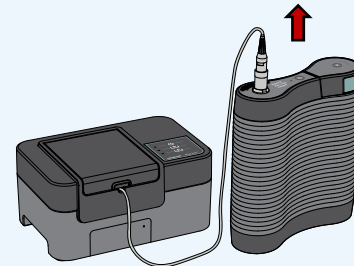

MyLink


Optune Gio  
device

4

Once download is complete, the middle  light will turn solid green and MyLink will beep. **Unplug the gray MyLink cable** from the Optune Gio device.



5

When the bottom  light is blinking, that means data is uploading to Novocure's system. This can take up to 15 minutes. Once all three lights are solid green, the upload was successful. **You can turn off MyLink.**